

How to pair your hearing aids with your Android™ device

1. Go to **Google Play**. In the **Search** field on top, enter the name of the app and download the app.
2. Once downloaded, open the app and tap **Get Started**.
3. Follow the instructions on the screen to connect. You are now ready to go.

NOTE: If you have previously paired any hearing aids, you need to unpair them in your smartphone's Bluetooth menu before you pair them again.

If you need to re-pair your iOS device

1. Start by removing the old pairing by going to the **Settings** menu.
2. Scroll down and select **Accessibility**.
3. Select **Hearing Devices**
4. Tap on your hearing aids, scroll down and tap on **Forget this device**.
5. Turn your iOS device off and on again and re-pair the hearing aids to the device. If you need more information, contact your hearing care professional.

Worldwide Headquarters
Belton A/S
Lautrupbjerg 7
DK-2750 Ballerup, Denmark
Tel.: +45 45 75 11 11
beltone.com

CVR no. 55082715



© 2019 GN Hearing Care Corporation. All rights reserved. Belton is a trademark of GN Hearing Care Corporation. Apple, the Apple logo, iPhone, iPad and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Android, Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc.

M201464GB-19.08-Rev.C




Get connected

The Belton HearMax™
and Belton HearPlus™ apps



Belton.com

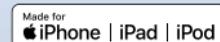

Introduction

Thank you for choosing our products. We have created this app to make it easy and pleasant for you to use and control your hearing aids from your smartphone or mobile device.

For information on which smartphones and mobile devices are compatible with the app, visit: beltone.com/compatibility

Before you can start using the app, you must pair your smartphone or mobile device with your hearing aids. This is easily done – just follow the steps below.

How to pair your hearing aids with your iOS device

1. Turn your hearing aids off.
2. Turn on **Bluetooth**® by going to the **Settings** menu on your iOS device. The slider turns green when you activate Bluetooth.



3




4

Note: From iOS 13 and newer.

3. Go back to the main **Settings** menu and then, tap on **Accessibility**.
4. Scroll down to **Hearing Devices**.

5. Turn your hearing aids on. Now, your iOS device will be able to detect your hearing aids.

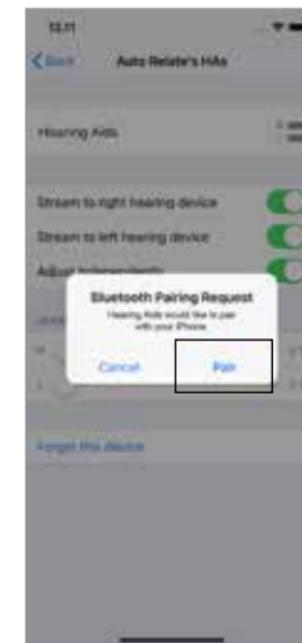

6

6. Tap on **Pair**.
7. Go to the **App Store**, and in the **Search** field on top enter the name of the app.
8. Download the app, open it and tap **Get Started**.
9. Follow the instructions on the screen to connect to the app. You are now ready to go.

